



Instructions
for Use Personal Prophylactic Hygiene Care
Female Liquid Vaginal Suppositories
SCHALI® Care-FS
Adjuvant Gynaecological Surgery

Instructions for Use
Personal Prophylactic Hygiene Care
Female Liquid Vaginal Suppositories
SCHALI® Care-FS
Adjuvant Gynaecological Surgery

SCHALI® Care-FS Liquid Vaginal Suppositories are a highly-effective photoactive product intended for treatment of vaginal mucosa in scope of complex therapy after gynaecological surgery and in postsurgical rehabilitation for better local regeneration and protection against bacteria and microbes.

PRODUCT FORM:

Prophylactic hydrogel in 3 ml disposable single-dose container.

INGREDIENTS:

Aqua, Hydroxypropyl Cyclodextrin, Glycerin, Propylene Glycol, Glycereth-18 Ethylhexanoate, Glycereth-18, Acrylates/C10-30 Alkyl Acrylate Crosspolymer, Titanium Dioxide, Triethanolamine, Diazolidinyl Urea, Methylparaben, Propylparaben, Phosphoric Acid.

MECHANISM OF ACTION:

Soft hydrogel formulation of the suppositories dissolves slowly in the vagina, where it is absorbed by vaginal mucosa. The product acts locally by inducing regeneration, strengthening vascular walls and improving microcirculation in capillaries, moistening and smoothing vaginal mucosa, reducing unpleasant sensation and inflammation (itching, burning pains, irritation). Plant and mineral ingredients and their constituents exert immunomodulating, anti-inflammatory, anti-aggregating, fibrinolytic, decongestant, thrombolytic and analgesic effects, resulting in lower transformation growth factors, better regeneration, better formation of connective tissue and physiological scar, and both faster healing and recovery.

Regular use of the product promotes generation of anti-inflammatory cytokines, regulation of immunoglobulins and antibodies, giving rise to a comprehensive immunomodulating effect, and to both induced phagocyte activity and interferon's genesis. In this way, local regeneration is stimulated, that causes hematomas and oedemas to resolve faster, vascular walls to restore normal penetrability, and both microcirculation and trophic processes to improve.

Mineral bio-constituent of the hydrogel formulation includes fragments of a certain class of metalloid semiconductors with unsaturated bonds - titanium dioxide crystalline lattice - known for its high catalytic activity. Photoactive particles of QD SCHALI® titanium dioxide trigger generation of active oxygen forms (singlet oxygen). Mechanism of action is based on the principle of energetic neutralization of free chemical bonds on the surface of toxic compounds by saturating them with complementary oxygen atoms, generated in the result of the catalytic reaction. Active oxygen forms may successfully treat most of all currently known exogenous and endogenous intoxications.

By inactivating unsaturated chemical bonds in peptides, glycosides etc., titanium dioxide is capable to remove harmful infectious toxins, resulting in a strong antimicrobial and antibacterial effect and in a quick relief of symptoms of different pathological conditions and inflammations, even in deeper vaginal tissues. Titanium dioxide induces vaginal epithelium cells to resist multiple infection and inflammation agents and in this way may prevent recurrent infection.

The product is intended for personal care purposes, has preventive action and may be used as complex therapy and photodynamic therapy agents in treatment of a wide range of gynaecological conditions. The product covers up and softly assuages genital mucosa, improves mucosal elasticity and flexibility. Regular use of the product induces regeneration of epithelial structures, regulates vaginal microflora and pH, and improves both secretion and quality of cervical mucus.

Hypoallergenic product.

INDICATIONS:

- For better vaginal mucosa regeneration in treatment of:
 - Dysplasia conditions (after surgery, cryo-destruction, laser destruction, diathermocoagulation etc.)
 - Postsurgical rehabilitation after vaginal cyst surgery
 - Vaginal fistula
 - Postpartum period
 - Birth injuries
 - Postcastrational syndrome
 - Ovarian abscess
 - Pelvic adhesive disease (morbus adhaesivus).

METHOD OF USE:

For vaginal use only.

Take one single-dose container out of the package with clean hands, insert into the vagina and press out its content. Apply one suppository 2 times a day for one month. Extend your course of treatment up to 90 days, if necessary.

SIDE EFFECTS:

No side effects found.

No product dependence and no rebound syndrome.

CONTRAINDICATIONS:

Do not use during pregnancy. No other contraindications detected.

STORAGE CONDITIONS:

Keep the product in a dark cool place at the temperature below 25°C.

SHELF LIFE:

Shelf life is 3 years from the date of production.

Do not use the product after expiration of the date of production, specified on the package.

SCHALI® Care Personal Hygiene Products are made under license No. 286-S.